

STEP 3B

TOXICOLOGY URINE SAMPLE

***USE ONLY FOR SUSPECTED
DRUG FACILITATED SEXUAL ASSAULT***

DO NOT PLACE URINE SAMPLE INSIDE KIT

Kit #: _____

Date Collected: _____ Time: _____ AM/PM

Collected by: _____

- IF INGESTION WAS WITHIN 120 HOURS, COLLECT URINE.
- IF INGESTION WAS OVER 120 HOURS, DO NOT COLLECT.

up to 90ml urine (up to 120 hours)

Urine

1. Collect urine in a sterile urine container (hospital supply) and seal in provided biohazard bag.
2. Place ice in second bag labeled ICE. Seal bag.
3. Place biohazard bag containing urine sample and "ICE" bag inside evidence bag and seal. Fill out labels accordingly.

Freeze or place on ice immediately upon collection to preserve sample.

Advise law enforcement to KEEP FROZEN.